

Instructions *Litho/Urology Table Drape*

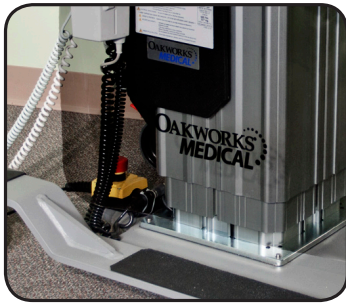


Fig. 1

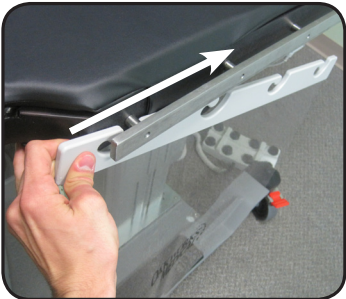


Fig. 2



Fig. 3




Fig. 4



Fig. 5

Instructions for installing the drape

1. Unfold drape from packaging.
2. Be sure Oakworks Logo is facing outward before attaching. (Fig. 1)
3. Slide one side of drape attachment onto the T-Rail supports on the table. (Fig. 2)
4. Slide other drape attachment onto opposite side of the table's T-Rail supports. (Fig. 3)

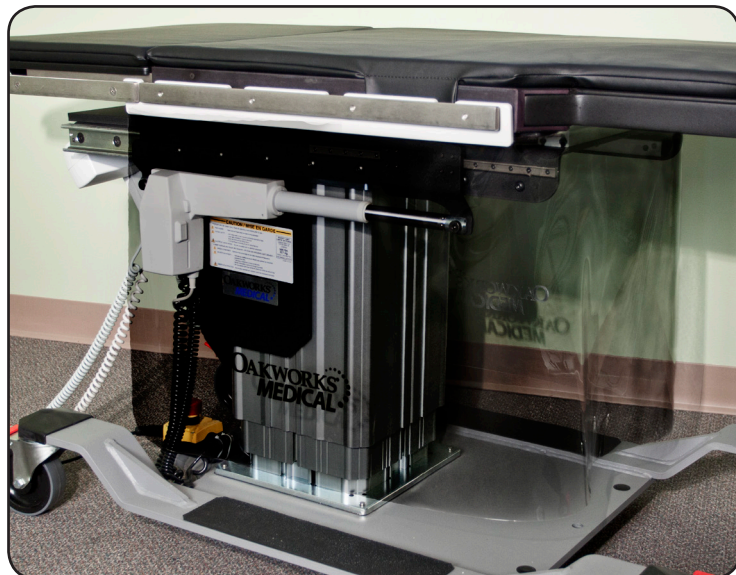

Cleaning Instructions

1. Slide one side of the drape attachment off the table's T-Rail support. (Fig. 4) Slide opposite side off and lay the drape on a flat surface with the Oakworks logo facing up. (Fig. 5)
2. Clean with a mild detergent.