


Set-up Instructions for the Portable Taping Table



Fig. 1

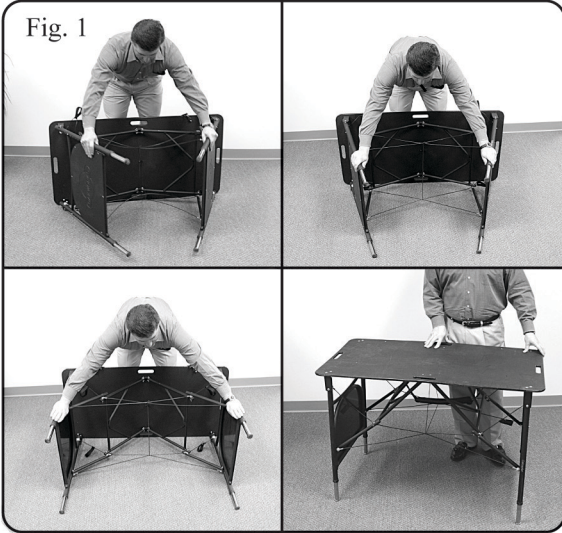

To Set-up:

- Fig 1 • Pull out the legs (as shown) until they lock into position.
- Set the table upright. The table ready for use.

To Fold Table for Transport:

- Fig. 2 • Pull quickly upward on the leg release strap located on the cables (as shown).
- Fig. 3 • Fold leg assembly neatly until closed. The table is ready to be transported or stored.

IMPORTANT!

- DO NOT allow anyone to rest their feet on any of the support structures. They are designed to be lightweight and strong, but are not to be used as foot rests. (Fig. 4)
- Use the Field Feet under the leg extensions to ensure a solid footing on soft or uneven ground. (Fig. 5) These should not be used indoors or on smooth surfaces, as the table may slide.

Fig. 2

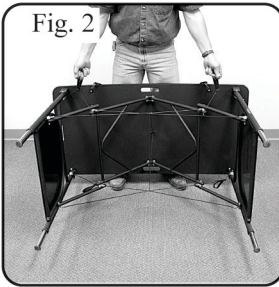


Fig. 3


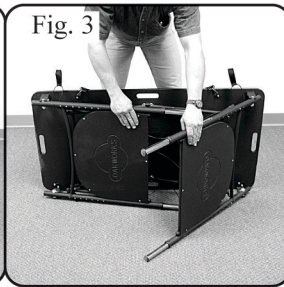


Fig. 4



Fig. 5

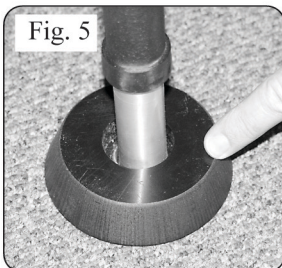

Cleaning & Care:

- The table is designed to be used indoors or, in fair weather, outdoors. If the table does get wet, wipe it dry to prevent rusting.
- For maximum strength, the leg extensions are made of steel which has been finished to prevent rusting. However, if leg extensions are allowed to become dirty, they can stick inside the leg and inhibit height adjustments.
- The aluminum parts require no special care.
- Steel parts, pivot points and hinges benefit from an occasional drop of oil or a little teflon spray. Wipe off excess.
- Screws, nuts and other hardware should be checked occasionally and tightened whenever loose.

OAKWORKS®

Phone: 717-235-6807
FAX: 717-235-6798
www.oakworks.com