

USER MANUAL - FLUORO EXTENDER

PRODUCT USE

The OAKWORKS® Fluoro Extender is radiolucent and offers extra width only where you need it to achieve greater positioning accuracy for optimum imaging.

SAFETY INSTRUCTIONS



DO NOT place undue weight or downward pressure. It is a positioning device for the arms or legs and should not be used as leverage to get on or off the table. Injury can occur.

DIRECTIONS FOR USE

Slide the Fluoro Extender under the table top pad.

Ask the patient to lie down on the table. Position the patient to suit the needs of the procedure. The weight of the patient will hold the Fluoro Extender in place under the table top pad.

When finished, ask the patient to rise up enough for you to remove the Fluoro Extender from under the table top pad.



CLEANING & DISINFECTION

Use appropriate sterilized field materials to isolate the patient from the Fluoro Extender. This also minimizes cleaning and disinfecting procedures. Use a 10% sodium hypochlorite (bleach) solution or *Recommended Disinfectants* on all surfaces. Clean all sides of the extender. Follow the directions on the disinfectant and wipe off excess.


Recommended Disinfectants

Protex, MadaCide, Accell TB, Virox®

Note: Damage caused by unapproved substances will not be covered under the warranty.

SPECIFICATIONS

Dimensions (X Y Z)	30" x 23-1/2" x 1/4" (76cm x 60cm x .6cm)
Maximum Weight Rating	30 lbs. (13.6 kg)
Radiolucency	Maximum of .85mm AL @ 100 kVp. HVL of 3.6mm
Environmental Conditions	Temperature: 50°F (10°C) to 100°F (38°C) Humidity: 20% to 60% RH Atmospheric Pressure: 98 to 105 kPa

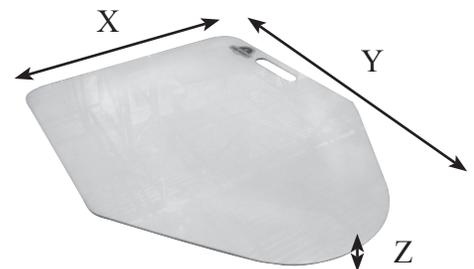

MAINTENANCE & SERVICE

No maintenance required

REPLACEMENT PARTS

- PN 2005: **Fluoro Extender**