

How to Care for Your Fabric

CLEAN your upholstered items when needed using a mild solution of 4:1 diluted non-alcohol cleaner such as Formula 409®, Fantastik®, or Green Windex®

DISINFECT your fabric with Protex, MadaCide, MadaWipe, Accell TB, Virox® (Canada), or a diluted 1/100 bleach solution. Dry immediately after cleaning to remove excess cleaning solution.

DO NOT use citrus based cleaners or other strong cleaners, such as alcohol, acetone, higher concentrations of bleach or other products that contain high concentrations of these substances.

DO NOT expose the fabric to temperatures below 32°F/0°C or above 95°F/35°C.

DO NOT expose the fabric to direct sunlight, adhesives, liquids, or abrasive materials.

**FAILURE TO FOLLOW RECOMMENDATIONS
WILL VOID THE WARRANTY ON THE FABRIC.**

*If you have any questions, call Customer Service
at 717-235-6807 or visit our website:
www.oakworks.com*